SPECIFICATIONS

Dimensions: 140 mm x 173 mm x 135 mm (without chamber fitted)

2.8 kg (no chamber fitted), 3.1 kg (chamber fitted and filled with water) Weight: 50/60 Hz Supply Voltage: 115 V~ Supply Current: 2.0 A max at 115 V~ Supply Frequency:

22 V~, 2.73 A, 60 W, 50/60 Hz Heater Plate: 150 W Heater Wire:

Heater Plate Over-temperature Cutout: 118 ± 6 °C

Temperature Control Settings

Chamber outlet: 35.5 - 42 °C, Airway 35 - 40 °C Chamber outlet: 31 - 36 °C, Airway 28 - 34 °C Invasive Mode: Noninvasive Mode:

Three digit 14 mm 7 segment LED Display:

10 - 70 °C, Accuracy: ± 0.3 °C (in 25 - 45 °C temperature range) Range:

Alarm Parameters

High Humidity Alarm: An immediate, audible alarm at a displayed temperature of 41 °C or if the

airway temperature exceeds 43 °C

An audible alarm between 10 minutes @ 29.5 °C, and 60 minutes @ 34.5 °C Low Humidity Alarm:

(Invasive Mode only)

Sound Pressure Level: Alarms exceed 50 dBA @ 1 m

Performance

Recommended ambient temperature range: 18 - 26 °C

CAUTION: If operating in ambient temperatures outside the recommended range, consult your local

Fisher & Paykel Healthcare representative.

Recommended Flow Range: Invasive Mode: up to 60 L/min, Noninvasive Mode: up to 120 L/min

Refer to breathing circuit specifications for minimum flow **Humidity Performance:** Invasive Mode: > 33 mg/L, Noninvasive Mode: >10 mg/L Maximum Operating Pressure: Refer to chamber and breathing circuit specifications

Warm-up time: Less than 30 minutes

Symbol Definitions











Alternating to vertica



falling drips

- Hot









Underwriters Laboratories Inc.
with respect to electric shock, fire
and mechanical hazards only
in accordance with UL 60601-1 and CAN/CSA C22.2 No. 601.1 84DJ

WARNING

- Ensure that invasive mode is set for patients that have bypassed airways.
- The use of breathing circuits, chambers or other accessories which are not approved by Fisher & Paykel Healthcare may impair performance or compromise safety.
- Ensure that both temperature probe sensors are correctly and securely fitted. Failure to do so may result in temperatures in excess of 41 °C being delivered to the patient.
- Ensure maintenance of grounding integrity by connection to a "hospital grade" receptacle.


 Always disconnect supply before servicing.
- When mounting a humidifier adjacent to a patient ensure that the humidifier is always positioned lower than the patient.
- operation of high frequency surgical apparatus, shortwave or microwave equipment in the vicinity of the humidifier may adversely affect its function. If this occurs, the humidifier should be removed from the vicinity of such devices.
- Do not touch the glass tip of the chamber temperature probe during use. Keep black connectors dry at all times.
- Check accessories for any physical damage before use and replace if damaged.
- California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: www.fphcare.com/prop65

Rx only

CLEANING *MR850 Heaterbase* Using a damp cloth, clean the humidifier with either of the following: Isopropyl Alcohol, normal dishwashing detergent. *Temperature Probe* Always clean before use. Probes can be cleaned with one of the following solutions: Sporicidin™, Cidex™ OPA, Sporox™ *OR* sterilize the probe using Ethylene Oxide sterilization at 55 °C (131 °F), 80 kPa, (allow at least 15 hours for residual ETO to disperse before use). Wipe the airway temperature probes clear of any cleaning residues before use. Store probes in clean conditions. DO NOT autoclave probes.

CAUTION: DO NOT immerse the heaterbase or temperature probe electrical connections in any liquid.

NOTE: Follow the cleaning agent manufacturers' instructions carefully. It is the user's responsibility to qualify any deviations from these procedures, both for disinfecting efficacy and physical effect on the probe and the heaterbase. See the MR850 technical manual for more detailed instructions on cleaning.

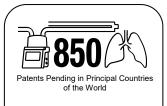

INSTRUCTION SHEET DO NOT DISCARD

MR850 RESPIRATORY HUMIDIFIER



The MR850 respiratory humidifier is used to warm and humidify gases delivered to patients requiring mechanical ventilation, positive pressure breathing assistance, or other medical gases.

185042343 Rev K 2016-05 © 2016 Fisher & Paykel Healthcare Limited

MANUFACTURER 🕍 Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348 Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.co.nz Web: www.fphcare.com

AUSTRALIA (Sponsor) Fisher & Paykel Healthcare Pty Limited, 36-40 New Street, PO Box 167, Ringwood, Melbourne, Victoria 3134, Australia Tel:+61 3 9879 5022 Fax:+61 3 9879 5232 AUSTRIA Tel: 0800 29 31 23 Fax: 0800 29 31 22 BENELUX Tel:+31 40 216 3555 Fax:+31 40 216 3555 Fax:+31 40 216 3554 BRAZIL Fisher & Paykel do Brasil, Rua Sampaio Viana, 277 cj 21, Paraíso, 04004-000, São Paulo – SP, Brazil Tel: +55 11 2548 8002 CHINA 代理人/售后服务机构:费雪派克医疗保健(广州)有限公司,广州高新技术产业开发区科学城科丰路31号G12栋301号 电话: +86 20 32053486 传真: +86 20 32052132 FINLAND Tel:+358 (0)405 406618 Fax:+46 (0)8 36 6310 FRANCE Tel:+33 1 6446 5201 Fax:+33 1 6446 5221 GERMANY Tel:+49 7181 98599 0 Fax:+49 7181 98599 66 INDIA Tel:+91 80 4284 4000 Fax:+91 80 4123 6044 IRISH REPUBLIC Tel: 1800 409 011 ITALY Tel: +39 06 7839 2939 Fax: +39 06 7814 7709 JAPAN Tel: +81 3 5117 7110 Fax: +81 3 5117 7115 KOREA Tel: +82 2 6205 6900 Fax: +82 2 6309 6901 NORTHERN IRELAND Tel: 0800 132 189 RUSSIA Tel and Fax: +7 495 782 21 50 **SPAIN** Tel:+34 902 013 346 Fax:+34 902 013 379 **SWEDEN** Tel: +46 8 564 76 680 Fax: +46 8 36 63 10 **SWITZERLAND** Tel: 0800 83 47 63 Fax: 0800 83 47 54 **TAIWAN** Tel:+866 <u>2 8751 1</u>739 Fax:+886 2 8751 5625


TURKEY Fisher Paykel Sağlık Ürünleri Ticaret Limited Şirketi, Alinteri Bulvari 1161/1 Sokak No. 12-14, P.O. Box 06371 Ostim, Ankara, Turkey Tel: +90 312 354 34 12 Fax: +90 312 354 31 01 UK Ltd, Unit 16, Cordwallis Park, Clivemont Road, Maidenhead, Berkshire SL6 7BU, UK Tel:+44 1628 626 136 Fax:+44 1628 626 146 USA/CANADA Tel: 1800 446 3908 or +1 949 453 4000 Fax:+1 949 453 4001

(us)

SET UP

- Slide humidification chamber (A) onto humidifier base and connect breathing circuit (B) (refer to humidification chamber and breathing circuit operating instructions for further details).
- 2. Connect the temperature probe plug (C) (REF 900MR86X) to the blue socket on the humidifier base until an audible click is heard.
- 3. Push the chamber probe (**D**) and airway probe (**E**) into the breathing circuit. Make sure the chamber probe is correctly located in its key-way and that both probes are pushed home. The probe lead can be restrained using breathing circuit clips.
- 4. Connect the heater wire adaptor plug (F) (REF 900MR8XX) to the yellow socket on the humidifier base until an audible click is heard.
- 5. Connect the other end(s) of the heater wire adaptor to the breathing circuit socket(s) (G) and (H).
- 6. The humidification system is now set up and ready for use. After power on, the humidifier will default to invasive mode.

ATTENTION: Refer to operating instructions which accompany each accessory.

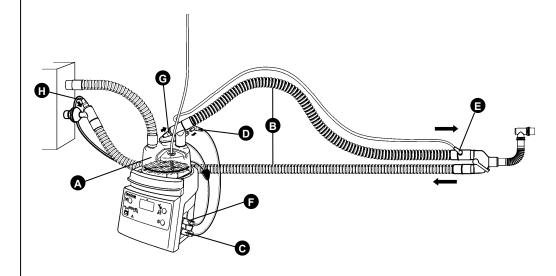

THE FOLLOWING ACCESSORIES ARE REQUIRED:

- Humidification Chamber (e.g: MR290)
- Breathing Circuit (e.g: RT110)
- Temperature Probe (e.g: 900MR869)
- Heater Wire Adaptor (e.g: 900MR800)
- Mounting Bracket (to suit ventilator)

Choice will depend upon application. Please contact your local Fisher & Paykel Healthcare representative for recommendations.

OPERATION

MUTE

The mute button silences the humidifier's audible alarm for at least two minutes. The muted time depends on the alarm condition and the severity of its cause.

SET-UP INDICATORS



Chamber & Airway Probes

Lights if either the chamber probe or the airway probe is not inserted into the breathing circuit correctly.



Heater Wire

Lights if the heater wire adaptor or breathing circuit has not been connected, or is damaged.



Temperature Probe

Lights if the temperature probe is not correctly plugged into the MR850, or the probe is faulty.



Water Out

Lights when there is insufficient water in the chamber. Check water supply. Maximum time to alarm of 20 minutes.



See Manual

The humidifier and all accessories should be immediately replaced and sent for servicing.

TEMPERATURE DISPLAY

Displays the saturated gas temperature (the lower of the airway and chamber temperatures in $^{\circ}$ C) delivered to the patient. This display will normally show the chamber temperature (around 37 ± 0.5 $^{\circ}$ C for invasive mode, and 31 ± 0.5 $^{\circ}$ C for noninvasive mode).

By pushing and holding the mute button for one second, the chamber outlet temperature then the airway temperature is displayed. The display will then revert to normal operation.



LOW TEMPERATURE ALARM

An audible alarm and flashing temperature display showing **35.5** °C or lower (in invasive mode only). Caused by cold/drafty conditions or very high or low gas flows.

If the circumstances causing the low humidity alarm cannot be changed then the audible alarm acts as a reminder that the patient is receiving inadequate humidity and may require further intervention to maintain airway clearance.

HIGH TEMPERATURE ALARM

A flashing temperature display showing **41 °C or higher**. The humidifier will discontinue heating of the chamber and circuit until the temperature decreases to within normal limits.

MODE BUTTON

This button switches between invasive and noninvasive mode.



Invasive Mode

This mode is for patients with bypassed airways. The humidifier delivers gas as close to body temperature saturated (37 °C, 44 mg/L) as possible.

Under cold or drafty conditions the chamber temperature may drop as low as 35.5 °C in order to maintain a dry breathing circuit.

WARNING: Ensure that invasive mode is set for patients that have bypassed airways



Noninvasive Mode

This mode is for patients receiving face mask or headbox therapy, and delivers a comfortable level of humidity.

ON/OFF BUTTON

The humidifier will power ON if this button is held down briefly, but must be held down for one second to turn the humidifier off. The humidifier will always default to invasive mode when it is turned on.

ROUTINE MAINTENANCE

Refer maintenance to qualified service personnel. A full technical description including routine maintenance and service data is contained in the Technical Manual which is available from your supplier or Fisher & Paykel Healthcare (REF 185041340)

NON-HEATED BREATHING CIRCUITS SET UP

- Slide humidification chamber (a) onto humidifier base and connect breathing circuit (b) (refer to humidification chamber and breathing circuit operating instructions for further details).
- 2. Connect the temperature probe plug (c) (REF 900MR86X) to the blue socket on the humidifier base until an audible click is heard.
- 3. Push the chamber probe (d) and airway probe (e) into the breathing circuit. Make sure the chamber probe is correctly located in its key-way and that both probes are pushed home. The probe lead can be restrained using breathing circuit clips.
- The humidification system is now set up and ready for use. After power on, the humidifier will default to invasive mode.

ATTENTION: Refer to operating instructions which accompany each accessory.

RECOMMENDED SETUP FOR ADULT PATIENTS

Non-heated breathing circuit with MR290 Chamber 900MR869 Temperature Probe, Mounting bracket to suit ventilator

For all other applications refer to Product Guide or contact your local Fisher & Paykel Healthcare representative.

SPECIFICATIONS (additional)

Temperature Control Settings (non heated circuits)

Invasive Mode: Airway 37 °C Noninvasive Mode: Airway 31 °C

Alarm Parameters (non heated circuits)

High Temperature Alarm: Immediate, audible alarm at a displayed

temperature > 41 °C


Low Temperature Alarm: Audible alarm at a displayed temperature

< 29 °C (Invasive), < 26 °C (Noninvasive)

Performance (non heated circuits)

Recommended Flow Range: Invasive Mode: 6 to 60 L/min

Non-Invasive Mode: 6 to 120 L/min

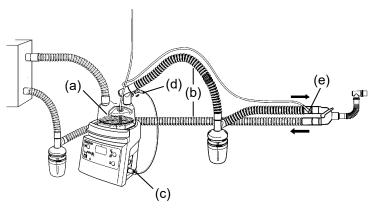

NOTE

To set the humidifier to non-heater wire mode, refer to the MR850 technical manual. (Rev. H or later).

If a heated breathing circuit is used while in non-heater wire mode the humidifier automatically changes to heater wire mode. To use non-heater wire mode again will require it to be manually set.

The airway probe out test does not work with non-heated breathing circuits. A low humidity alarm will instead be given.

When there is no gas flow the humidifier will give a low humidity alarm

If the chamber runs dry either a water out alarm or a low humidity alarm will occur.